Pure Care

After-bleaching care

FR

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EN

1 INTENDED USE

Pure Care is a paste to be applied on the teeth for post-treatment after a bleaching procedure.

2 PRODUCT DESCRIPTION AND USER

2.1 Product specific description

Pure Care is a water-based paste that reduces tooth sensitivity after teeth have been bleached.

2.2 Patient target group

To be used on all patients whose teeth have just been bleached.

2.3 User

The Pure Care is used by the dentist in the dental practice directly on the patient after a bleaching treatment.

3 COMPOSITION

Purified Water, Xylitol, Glycerin, Potassium Nitrate, Sodium Fluoride, Thickener, Flavourings

4 INDICATIONS

For after-treatment of the teeth after a bleaching procedure.

5 CONTRAINDICATIONS

If the patient is allergic or hypersensitive to one of the components, this product must not be used or only under the strict supervision of the treating doctor/dentist.

6 INTERACTIONS WITH OTHER AGENTS

Well-known cross-reactions or interactions of the medical device with other materials already in the mouth must be taken into account by the dentist when using it.

7 APPLICATION / PROCESSING

7.1 Application with a custom tray

- After bleaching, clean the customized tray (or deep-drawn splint) under running water.
- Fill Pure Care into the tray and apply to the upper and lower jaw.
- Leave the tray in the patient's mouth for about 3 to 5 minutes.
- Remove the tray.
- Ask the patient to rinse their mouth well. The patient should not eat or drink for at least 30 minutes after the treatment.
- Clean the tray from remaining Pure Care by rinsing it under running water or brushing it out.

7.2 Application without a tray

- If necessary, dab excess saliva from tooth surfaces.
- Apply a generous amount of Pure Care to the tooth surface using a disposable brush, glove or, in difficult interproximal situations, an interproximal brush.
- Leave Pure Care in the patient's mouth for approx. 3 to 5 minutes.
- Ask the patient to rinse their mouth well. The patient should not eat or drink for at least 30 minutes after the treatment.

8 NOTES ON STORAGE AND HANDLING

Store at 3 - 25 °C.

9 SHELF LIFE

The maximum shelf life is printed on the label of each syringe. Do not use after the expiration date.

10 SIDE EFFECTS

Undesirable side effects of this medical product are extremely rare if processed and used properly. In principle, however, immune reactions (e.g. allergies) or local discomfort cannot be completely ruled out. All serious incidents occurring in connection with the use of this product must be reported to the manufacturer specified below and to the relevant competent authority.

11 INSTRUCTIONS FOR DISPOSAL

Residual quantities and packaging material must be disposed of in accordance with local and/or statutory regulations.

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