# Dentoetch

## FR

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## EN

Etching gel - Phosphoric acid 37%. DENTOETCH contains phosphoric acid for etching tooth enamel and dentin when preparing surfaces for the application of composites.

### PROPERTIES:

High quality, non-drying, medium viscosity gel. Stays put, does not run. Comfortable-to-use syringe. Tip can be angled to facilitate easy accurate placement of the gel even in distal & lingual composite preparations. Washes off quickly & easily. Has excellent water solubility. Well visible blue color. Disposable tips save time and are more sanitary.

### DIRECTIONS FOR USE:

The effective recommended etching time is: Enamel: 30 to 60 seconds. Dentin: 15 seconds. The DENTOETCH should then be rinsed off with copious quantities of water for 30 seconds. The disposable tip should be removed and discarded after use. Recap syringe when not in use.

### DRYING:

Enamel: dry adequately until the surface has a frosty white appearance. Dentin: Dry moderately.

### CAUTION:

DENTOETCH contains phosphoric acid. Avoid contact with skin and eyes. If accidental contact occurs, flush with copious amounts of water. If swallowed, drink a large amount of water/milk and consult a doctor, do not induce vomiting.

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