Aireo V-Lock System

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
EN

DESCRIPTION

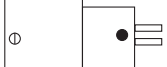AIREO V-LOCK SYSTEM is a disposable flexible tip for air/water syringes. The tip has two independent channels to guide the air and water from the syringe to the end of the tip. This product is compatible with the dental air/water syringe adaptors indicated below:

The AIREO V-LOCK SYSTEM tip is packaged in a box of 200 units, available in 3 colors (green, blue, and white).

COMPOSITION

- Punta azul : Medical grade polyethylene (97%), blue coloring agent (3%)
- White tip : Medical grade polyethylene (97%), white coloring agent (3%)
- Green tip : Medical grade polyethylene (98%), white coloring agent (2%)

INDICATION

AIREO V-LOCK SYSTEM is intended for use in providing optimal tooth rinsing/drying.

CONTRAINDICATIONS

There are no contraindications to the use of this device.

TARGET POPULATION

All patients, from the appearance of their first teeth, who require water cleaning or air drying of teeth during a dental procedure.

USERS

This product is intended solely for use by dental professionals.

CLINICAL BENEFITS

Eliminate the risk of cross-contamination.

INTENDED USE

AIREO V-LOCK SYSTEM can be used to spray dry air, water, or both at once. Using AIREO V-LOCK SYSTEM also eliminates the risk of cross-contamination, since the tip is disposable. AIREO V-LOCK SYSTEM allows users to access hard-to-reach places.

INSTRUCTIONS FOR USE

Preparation

- 1) Push the tip onto the adaptor. The notch ensures correct orientation of the tip.
- 2) Ensure that the tip is firmly in place by testing it under air pressure before using it on the patient.

Using the device

- 3) During use, the tip can be bent for improved access inside the mouth.

After use

- 4) After use, remove the tip from the syringe.
- 5) Dispose of the tip according to these instructions.

SAFETY PRECAUTIONS

Disposable product: Do not reuse. Risk of cross-contamination between patients if reused.

SIDE EFFECTS OR ADVERSE EFFECTS

There are no side effects or adverse effects to report in connection with use of this device. Any serious incident occurring in connection with the device should be reported to the manufacturer and to the competent authority in the member state where the user and/or patient are located.

RESIDUAL RISK

No residual risk is associated with this device.

PERFORMANCE CHARACTERISTICS

Optimal propulsion of dry air and/or water.

STORAGE CONDITIONS

Keep away from sunlight.

PRODUCT DISPOSAL

Dispose of the product in accordance with DASRI regulations on biological waste. Do not dispose in the environment.

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