# Dentoclic Gates & Largo

## FR INDICATION

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## EN

### INTENDED USE

*Gates & Largo are drills used to widen and straighten the coronal third of the canals in order to provide improved access for instruments used to work on the canals. Their use is limited to the right section of the canals.*

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Medical grade stainless steel

## PRECAUTIONS

- Drills must be used by professionals who have received adequate training
- Insert the Gates or Largo Drills into the contra-angle
- Drills should be used at moderate rpm speed (maximum 1200 rpm) and with a careful up and down movement only. If these precautions are not taken, drill can get stuck and/or break off in root canal.
- When performing a root canal treatment, it is of utmost importance that the root-canal has been prepared with a drill of appropriate size and shape. Always start with the smallest size of drill and work up to the size required.

## CONTRA-INDICATION

No Contra-indication known till this date

**Storage** : Store dry and at room temperature.

## HYGIENE: CLEANING / STERILISATION PROTOCOL

### Recommendations

- Sterilisation of the items before their first use, or before each re-use, is the responsibility of the user. In the case where dirty and/or damaged items are used, ITENA CLINICAL cannot accept responsibility.
- The drills are meant to be used for a maximum of 10 cycles.
- Clean the instruments as quickly as possible after use.
- Do not use acid based detergent solutions, abrasive cleaners, nor chlorine.

### Manual cleaning

- Clean the instruments with an appropriate nylon brush in order to remove the smear layer. There must not be any further traces of dirt on the items.
- Immerse the items, in a disinfectant solution which is suitable for medical items\* for 15 minutes.
- Check that there are no traces of dirt on the items. If necessary, repeat the cleaning and disinfection cycle.
- Rinse the items thoroughly in demineralised or distilled water to avoid corrosion and alterations during sterilisation. Dry in compressed air for 2 minutes. Visually verify the cleanliness, and remove deformed instruments. If the instruments are still dirty, repeat the cleaning procedure.

### Automatic cleaning

- Use a cleaning and disinfectant appliance, sorting the instruments by type and category of the materials to avoid electrolysis and corrosion events. Place the items fully submerging them, and start the cleaning cycle as per the following instructions:

Stages	Type of product and concentration	Time/Temperature
Pre-washing	/	/
Washing	Disinfectant solution, suitable for the medical items*	15 minutes at 55°C
Rinsing	Demineralised water	2 minutes at room temperature (15-25°C)
Drying	/	According to the autoclave settings

- At the end of the cycle, check that there are no signs of dirt on the items If necessary, repeat the cycle.

### Sterilisation

- Place the items in the appropriate sterilisation package.
- Place the package in the autoclave, and carry out one sterilisation cycle: 134°C at 2.2 bars for 18 minutes.
- Store the items away from all sources of contamination.

### Infectious risk:

To avoid the risk of infection, follow the sterilization instructions.  
The used product must be disposed of with biohazardous waste.

\* Based on ethyl alcohol and small concentrations of potassium hydroxide and ethoxylated alcohol.

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