Dentoclic Scan Post

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LEGEND OF LABELS AND SYMBOLS

Explanation of labels and symbols.

	CAUTION Important notice that must be strictly observed
	Date of manufacture
	Manufacturer
	CE marking
	LOT number
	Consult Instructions for Use
	Model number

	Medical Device
	The product is delivered non-sterile
	Allowed reprocessing cycles
	Distributor
	Keep away from sunlight

INTENDED USE, INDICATION FOR USE, AND CONTRA-INDICATIONS

The Dentoclic Scan Posts are geometry indicators required for scanning post-and-core cases. For intra-oral scanning, the posts are placed in the patients' root canal(s). For model and impression scanning, the scan posts are placed into those, in holes created by impression posts.

5.1.2 INDICATION FOR USE

The Dentoclic Scan Posts are indicated for:

- Dental Prosthetic Restorations
- Endodontic Treatments

5.1.3 INTENDED USER PROFILE

The Dentoclic Scan Post is operated by trained dental professionals and qualified personnel.

5.1.4 INTENDED PATIENT PROFILE

Patients with the need for endodontic and prosthetic treatment with no restrictions to patient population.

5.1.5 CONTRA-INDICATIONS

None

IN PATIENT USE – STEP BY STEP

STEP 1: PRESELECTION OF THE SCAN POST

- Select the appropriate variant and length of the scan post.
- Ensure that variant matches the used Post & Core drill system and size.
- Ensure that length is appropriate, the "head" of the post (the cylindrical body) should not be in contact with teeth or other scan posts when mounted as this can cause bending and/or deformation.

STEP 2: PREPARATION FOR INTRA-ORAL USE

Preparation for use

To avoid cross contamination between patients or contamination of operators, the scan posts must be cleaned and sterilized prior to each use.

Cleaning procedure

- Clean the posts by rinsing with clean, preferably purified, water until gross soil is removed.
- Fully immerse in a non-enzymatic neutral pH detergent, e.g., Intercept™ and brush the scan post with a soft bristle brush for 1 minute.
- Rinse with clean, preferably purified, water for min. 2 minutes
- Dry the articles with a clean lint-free cloth

Sterilization procedure

After cleaning, the scan posts must be autoclaved prior to use.

- Clean Scan Posts are sealed in appropriate pouches. Refer to your autoclave's instructions for use on which pouches to use.
- Pouches are placed in sterilizer.
- Autoclaving Cycles is completed, recommended cycles in table below.

Description	Cycle Option A	Cycle Option B
Temperature:	121°C	134°C
Full Cycle Exposure Time:	15 minutes	3 minutes
Dry Time:	20 minutes	20 minutes

- After autoclaving, let the scan posts cool to room temperature

	The product is delivered non-sterile and should be cleaned and sterilized prior to every use.
---	---

	WARNING Replace the Scan Post if any damage, i.e., scratches, deformation, excessive wear, etc., is determined.
	WARNING The sterile barrier must be maintained until use!
	CAUTION If still hot, skin contact may cause discomfort to the patient. The scan post must not be warmer than body temperature during used.

STEP 3: POSITIONING AND SCANNING

- Insert the impression post into the patient's prepared root canal(s) and confirm that satisfactory fit and seating is obtained.
- Confirm that the appropriate length of the scan post has been chosen and that the scan post is included in the digital library. If required, follow the scanners instructions for use to download the appropriate library. The "head" of the post (the cylindrical body) may not be in contact with teeth or other scan posts as this can cause bending.
- Scan the intraoral situation following the scanner's instructions for use. Make sure the head of the posts scanned fully and without holes, both top and sides.

	WARNING To avoid aspiration or swallowing of the Scan Post during intra-oral scanning, ensure that the scan post has secure fit and is seated correctly. In case of uncertainty, it is recommended to do an x-ray confirmation of seating and fit.
	WARNING To avoid aspiration or swallowing extreme caution should be exercised when using pressurized air in the oral cavity.
	WARNING Use dental suction to collect blood and saliva during dental procedures.

	<p>CAUTION</p> <p>The standard recommendation for handling small instruments intra-orally is to tie a 20 cm long cord (Dental Floss) around the device, ensure to fix it firmly in place and position the patient in such a way that, should the scan post be dropped, it will not fall into the patient's respiratory or digestive tract. In the case of the scan post being dropped, it can be securely retrieved using the cord to pull it out.</p>
	<p>CAUTION</p> <p>To avoid aspiration or swallowing despite all above preventive measures, dental suction equipment can be used for recovering a dropped device. In case of swallowing or inhaling the scan post, follow standard procedures and contact the nearest medical facility.</p>

STEP 4: UNMOUNTING THE SCAN POST

Unmount the Scan Post by grasping it tightly in a secure pinch grip between thumb and index finger retracting it from the model in a linear motion.

If the scan post has wedged itself and is difficult to remove the situation can be mitigated by introducing a slight momentum (rotate).

	<p>WARNING</p> <p>Avoid applying excessive torque to scan post during retraction as this may cause unnecessary deformation or even break the Scan Post.</p>
	<p>WARNING</p> <p>It is not recommended to use tools during the mounting, retraction or dislodging as this may cause damage, deformation or even break the Scan Post.</p>
	<p>WARNING</p> <p>Use dental suction to collect blood and saliva during dental procedures.</p>
	<p>CAUTION</p> <p>To avoid aspiration or swallowing despite all above preventive measures, dental suction equipment can be used for recovering a dropped device. In case of swallowing or inhaling the scan post, follow standard procedures and contact the nearest medical facility.</p>

STEP 5: CLEANING AND STORAGE

Following use, the scan posts must be cleaned prior to storage.

Cleaning procedure

- Clean the scan posts by rinsing with clean, preferably purified, water until dust or debris is removed.
- Fully immerse in a non-enzymatic neutral pH detergent, e.g., Intercept™ and brush the scan post with a soft bristle brush for 1 minute.
- Rinse with clean, preferably purified, water for min. 2 minutes
- Dry the articles with a clean lint-free cloth
- Store in appropriate container.

	<p>WARNING</p> <p>The scan posts must be thoroughly cleaned immediately after use.</p>
---	---

USE WITH IMPRESSIONS AND MODELS

STEP 1: TAKING THE IMPRESSION

Choose a proper impression post that matches the system and diameter of the used drill. Also choose an appropriate length of the impression post. Insert the impression post into the patient's root canal(s). Take a regular impression. If the impression post is retained in the mouth, remove it.

	<p>CAUTION</p> <p>The Impression Post instruction for use should be observed for this step.</p>
	<p>CAUTION</p> <p>Always have an assistant in place with suction equipment ready for use. In case of swallowing or inhaling of devices or debris follow standard procedures and contact the nearest medical facility.</p>

STEP 2: MODEL CASTING

Make sure the impression post is in the impression, and then cast the model.

Remove the impression post from the model and replace it with an equivalent scan post (same length and diameter). The "head" of the post (the cylindrical body) may not be in contact with neighboring teeth on the model or other scan posts as this can cause bending

	<p>CAUTION</p> <p>Impression material instruction for use should be observed for this step.</p>
	<p>CAUTION</p> <p>Each Scan Post contains markings indicating the corresponding Post & Core systems, diameter, and length variant.</p>
	<p>CAUTION</p> <p>To avoid damage to the post and ensure that correct scan data is obtained, it is recommended that an in-depth knowledge of preferred systems is obtained prior to use.</p>
	<p>WARNING</p> <p>Replace the Scan Post if any damage, i.e., scratches, deformation, excessive wear, etc., is determined.</p>

STEP 3: SCANNING WITH THE SCAN POST IN PLACE

Scan the model following the scanner's instructions for use. Make sure the head of the scan posts are scanned fully and without holes, both top and sides.

	<p>CAUTION</p> <p>Ensure that the scan post has a secure fit and is seated correctly prior to scanning.</p>
	<p>CAUTION</p> <p>Evaluate the quality of the Scan prior to removing the Scan Post.</p>

STEP 4: UNMOUNTING THE SCAN POST

Unmount the Scan Post by grasping it tightly in a secure pinch grip between thumb and index finger retracting it from the model in a linear motion.

If the scan post has wedged itself and is difficult to remove the situation can be mitigated by introducing a slight momentum (rotate).

	<p>WARNING Avoid applying excessive torque to scan post during retraction as this may cause unnecessary deformation or even break the Scan Post.</p>
	<p>WARNING It is not recommended to use tools during the mounting, retraction or dislodging as this may cause damage, deformation or even break the Scan Post.</p>

STEP 5: CLEANING AND STORAGE

Following use, the scan posts must be cleaned prior to storage.

Cleaning procedure

- Clean the scan posts by rinsing with clean, preferably purified, water until dust or debris is removed.
- Dry the articles with a clean lint-free cloth
- Store in appropriate container.

	<p>WARNING The scan posts must be thoroughly cleaned immediately after use.</p>
---	--

STORAGE

The system storage box allows for sorting of scan posts by length and diameter.

	<p>The scan posts kit should be stored away from direct sunlight.</p>
---	---

LIFETIME

The lifetime of the product is dependent on several factors and it is recommended that the condition of the scan post is inspected prior to every use. The maximum amount of reprocessing cycles is secondary to the how progressed the general wear is.

	<p>Maximum 100 reprocessing cycles</p>
---	--

	<p>WARNING The scan posts may be damage and or change dimensions due to excessive wear and it is recommended that the Scan Posts are inspected prior to use and discarded at the first sign of wear or deterioration.</p>
---	--

DISPOSAL

Discarded items should be cleaned and disinfected prior to disposal to avoid cross-contamination. It is recommended to recycle them according to local regulations.

VIGILANCE

In case of any serious incident that has occurred in relation to the use of the device, please report the incident to CMT at contact@coremedtech.dk. You can also report it to the competent authority of the state in which the user and/or patient is established.

CONTACT INFORMATION

Contact your reseller for requests regarding support, customer service, software download or an upgrade. In case of malfunction, please report to your reseller or CMT at contact@coremedtech.dk.

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Internatikonal Associates Limited