

Dentocrown HD¹

Temporary restoration material

*Re-Discover
Dentocrown HD*



Setting time tailored

Material stability

Natural aesthetic appearance

Ergonomic gun

Enhance your
daily practice



Stable and natural crown

Key points

- Setting time: 2 min 30
- Material stability during chewing
- Natural aesthetic appearance
- Ergonomic gun

Mixing tips
& Thin intra-oral
tips



Mixing gun



Indication

Composite resin for making crowns, temporary bridges, inlays, onlays and veneers.

TIPS AND TRICKS

The Dentocrown HD can be reworked.

There are 2 methods available:

- > Remove the viscous layer (inhibition layer) from the surface of the temporary tooth with modified alcohol. Rework.
- > Use the temporary cement **Dentotemp** to fill in empty spaces.




Clinical case

Clinical case by Dr Bury Laurence




Initial view of tooth to be treated.

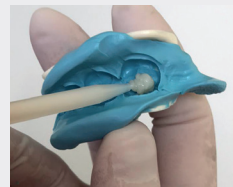

1 Fill impression with Dentocrown HD.



2 View of the impression.

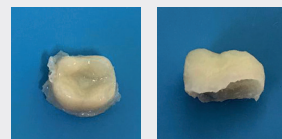

3 View of temporary tooth.



Final result.

Technical data*

- > High compressive resistance..... 330 MPa
- > Flexural strength..... 70 MPa
- > Preparation time..... 45 sec max
- > Time in mouth..... 45 sec

| Dentocrown HD | |
|---------------------------------|------------|
| Cartridges | |
| A1 74 g + 10 mixing tips | DWNHD50-A1 |
| A2 74 g + 10 mixing tips | DWNHD50-A2 |
| A3 74 g + 10 mixing tips | DWNHD50-A3 |
| Mixing tips¹⁶ | |
| Cartridge mixing tips | DWNE-10 |
| Mixing gun⁹ | |
| 50 ml 1:1 mixing gun | 153241 |



Enhance your daily practice

itena-clinical.com

Central Parc Bat B, 97 Allée de la Louve, 93420 Villepinte - FRANCE - T. + 33 1 45 91 61 40.
FP.DENTOCROWNHD.EN-B-04/2024

Follow us!



*Internal report, Itena Clinical, "Dentocrown HD".

¹⁶ Medical device Class IIa, CE 0425 - Manufacturer: Itena Clinical. ⁹ Medical Device Class I, Manufacturer: DXM CO, Ltd, South Korea. Authorized Representative: CMC Medical Devices & Drugs, Spain. ⁹ Medical Device Class I, Manufacturer: Medmix Switzerland AG, Switzerland. Authorized Representative: Medmix GmbH, Germany. Reserved for professional dental use, not reimbursed by health insurance organizations under the LPPR. Carefully read the instructions and labeling before use.