# Hydrospeed HD

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**EN**  
**INSTRUCTION FOR USE**  
- Classification : Vinyl Polysiloxane Impression Material (Dental use only)  
- Time in Minutes

Product Name	Room Temp. Working Time	Intraoral Setting Time
Hydrospeed HD Putty soft quick	30 sec	2 min
Hydrospeed HD Light Body quick	1 min	2 min 30 sec
Hydrospeed HD Bite registration	30 sec	1 min

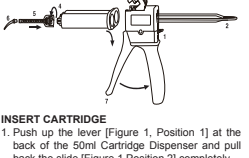
**DIRECTIONS FOR USE**  
1. Attach an intraoral tip to the mixing tip or load material into a syringe. Inject the mixture onto the prepared teeth.  
2. Load the tray with the appropriate material. (Putty or Tray)  
3. Seat the loaded tray in the mouth within the working time of each Consistency. Wait for the substance to set.  
4. After material has set, remove the impression and pour a model immediately.  
5. Mixing device is mounted on the cartridge

**STORAGE**  
Store in a dry and cool place (15-25°C)  
Shelf life : 2 years after production day  
**CAUTION**  
1. Avoid surface contact, or contamination with chemicals known to inhibit the hardening of the substance. (e.g. Polysulfide impression materials, Eugenol materials, Acrylates and Methacrylate residues, Latex rubber, and Sulfur compounds)  
2. Care should be taken to avoid getting the material on clothing.  
3. Close tubs immediately after use. Do not invert lids, and dosing spoons. (Putty)  
4. Do not use on patients who have a rash on their skin, dermatitis, or histories of hypersensitivity.  
5. Certain gloves and hand lotions inhibit the setting of vinyl polysiloxanes. (Putty)  
6. Test to insure compatibility before using the putty in case of accidental contact with eyes or skin. Rinse immediately with plenty of water and take appropriate action.  
7. Before use, check the expiration date presence of foreign substances and impurities.  
8. Use the manufacturer's manual to familiarize yourself with the compliance.

**INDICATIONS**  
- Putty & Light Body (Putty is formulated for use with the light body impression material to meet demands of the two viscosity techniques for multiple unit impressions).  
1. Impression of inlay, onlay, crown and bridge preparation.  
2. Functional impressions

- Bite Registration  
1. Intra-oral bite registration  
2. Bite registration for orthodontic needs  
**VIGILANCE**  
In case of any serious incident that has occurred in relation to the use of the device, please report the incident to Sell Global Co. at export@selglobal.co.kr and MedNet EC-REP at https://www.mednet-ecrep.com.

You can also report it to the competent authority of the state in which the user and/or patient is established.  
**USING THE CARTRIDGE**  
Figure 1

**INSERT CARTRIDGE**  
1. Push up the lever [Figure 1, Position 1] at the back of the 50ml Cartridge Dispenser and pull back the slide [Figure 1 Position 2] completely.  
2. Lift up the plastic lever [Figure 1, Position 3], insert the cartridge and lock it using the plastic lever [Figure 1, Position 3].  
**ATTACH MIXING TIP**  
1. Remove the cartridge cap [Figure 1, Position 4] or used mixing tip by turning it 90° counterclockwise, and discard it.  
2. Attach a new mixing tip [Figure 1, Position 5]  
3. Turn the new mixing tip clockwise 90° to lock it  
4. Attach an intraoral tip [Figure 1, Position 6].  
**APPLYING THE MATERIAL**  
- When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.  
- The material is mixed in the new mixing tip by pulling the trigger [Figure 1, Position 7] of the 50ml Cartridge Dispenser and it is then applied directly.  
**REMOVE CARTRIDGE**  
1. Push up the lever [Figure 1, Position 1] at the back of the 50ml Cartridge Dispenser and pull back the slide [Figure 1, Position 2] completely.  
2. Lift up the plastic lever [Figure 1, Position 3] and remove the cartridge.

**INDICATIONS**  
- Putty & Light Body (Putty is formulated for use with the light body impression material to meet demands of the two viscosity techniques for multiple unit impressions).  
1. Impression of inlay, onlay, crown and bridge preparation.  
2. Functional impressions

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