# Total C-ram

Self-adhesive resin cement

FR

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EN

**Self-etching & self-adhesive resin cement for ceramic**

TotalC-RAM is a dual-cure permanent adhesive resin cement specially formulated for ceramics cementation.

**INSTRUCTIONS FOR USE**

1. Remove the provisional restoration. Note: Use only temporary cement that are eugenol-free. The use of eugenol-containing temporary luting materials can inhibit the polymerization process of TotalC-Ram.

2. Rinse thoroughly.

3. Air dry the preparation (do not desiccate).

4. Try in the restoration to ensure proper fit.

5. Extrude a small quantity of material in order to align both pastes before use. Place a mixing tip on the auto-mix syringe.

6. Dispensing the material can be done as follows: Inlay/onlay. Mount the supplied intra oral tip onto the mixing tip and dispense the cement directly into the cavity preparation covering all surfaces. Seat the restoration gently onto the preparation allowing the cement to flow from all sides. Crown: Dispense the cement directly onto the preparation, covering all surfaces. Place the restoration gently on the preparation, allowing the cement to flow from all margins. Note: material working time is 1-2 minutes; Setting time is 3 minutes (at oral temperature) ...

7. When the restoration is properly set up, remove all excess cement. Excess cement is best removed in its gel state with scaler or explorer. Gel state can be achieved by light curing excess with a light unit for 2 seconds or allowing to self-cure for 1.5-2 minutes after application. For best results, leave to rest for 1.5 minutes before light curing.

8. After removal of excess cement, light cure all surfaces including margins for 20 seconds.

9. Use finishing strips for proximal areas. Polish margins with polishing discs, cups, or points.

10. Check occlusion.

| MAIN FUNCTIONS         | INGREDIENTS                                |
|------------------------|--------------------------------------------|
| Resin matrix           | TEGDMA, Bis-GMA, UDMA                      |
| Acid adhesive monomers | 4-META, meth-cycry phosphoric acid estersé |
| Coupling agent         | Silane                                     |
| Filling materials      | Nanosilice concentrate                     |

**CAUTION**

Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.

Note: Automix tips are designed for single patient use only.

**STORAGE AND SHELF LIFE**

Do not expose to temperature exceeding 24°C/75°F. Keep away from direct sunlight. Refrigerate for maximum shelf life. Do not freeze. If refrigerated, allow the syringe to reach room temperature before use. Do not use after expiration date. See expiry date on the packaging.

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