Micro-applicators

FR

Indications :

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EN

Indications

The micro applicators can be used for application of:

- Etching
- Bonding
- Hole lining
- Fluorine protective paint
- Hemostatic agent
- Pit and groove sealant, etc

Product Characteristics

Micro-applicators are available in conical and spherical shape. Applicator is divided into three sections, applicator head, applicator rod and breakable tail. The applicator rod is made of PP, which make it comfortable and elastic and can bend the applicator head to any angle position; the applicator head is made of high quality nylon hair.

Instructions for use

1. Take out the applicator and hold the rod in a comfortable position. Place the applicator head in the solution container and take the solution.
2. According to the coating position, bend the applicator head into any angles to make it easy to use.
3. Conical and spherical applicator can be selected according to the needs.

Storage and maintain method

1. The product shall be stored in a dry, ventilated and normal temperature room whose relative humidity is not more than 80% and doesn't have corrosive gas.
2. Avoidance of direct sunlight.
3. Prevention of extrusion and damage during storage.
4. Keep the package complete and avoid damaging the product by pressure

Contraindication

No contraindication is identified by now.

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Huanghua Promisee Dental Co., Ltd.
MedNet EC-REP GmbH