MTA Bioseal

Bioceramic root canal sealer

FR

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Bioceramic root canal sealer

INTRODUCTION :

MTA-BIOSEAL is a bioceramic root canal sealer based on MTA (mineral trioxide aggregate). Its composition provides the following qualities:

- Indicated for the permanent sealing of root canals;
- High radiopacity;
- Low expansion during setting;
- Low solubility in contact with tissue fluids;
- Optimal flow characteristics for insertion;
- Does not stain the tooth;
- Eugenol-free material: does not interfere with the setting of resin cements. CAUTION: Federal law restricts this device to sale by or on the order of a dentist.

COMPOSITION

- Base Paste: Salicylate Resin, Natural Resin, Calcium Tungstate, Nanoparticulated Silica, Pigments;
- Catalyst Paste: Diluting Resin, Mineral Trioxide Aggregate, Nanoparticulated Silica, Pigments.

INDICATION

MTA-BIOSEAL is a root canal sealer intended for the permanent sealing of root canals and may be used in combination with root canal filling materials.

CONTRAINDICATION

In patients with hypersensitivity against the resins or other components of the product.

WARNINGS

- MTA-BIOSEAL contains resins which may sensitize susceptible individuals. Do not use it in patients allergic to the resins or other components of the product;
- Avoid contact with eyes or skin. In case of contact, rinse immediately with water;
- Avoid contact with oral mucosa. In case of contact, rinse with water and prevent swallowing of product. In case any sensitivity persists, seek medical attention promptly;
- If the syringe becomes contaminated with saliva or blood during application, dispose of the syringe and do not use on an additional patient. Ensure that the lids of the base paste and catalyst are not switched, because switching them can cause hardening of the product inside the tube.

PRECAUTIONS

During the use of this product, it is recommended the patient and professional use suitable protective clothing, eye protection and gloves.

STORAGE.

- Keep in dry and cool place, away from humidity and sources of heat;
- Do not store in the refrigerator;
- Keep the product tightly capped.

TECHNIC OF USE

Root canal preparation: Prior to insertion of MTA-BIOSEAL, the root canal should be prepared and cleaned according to the selected endodontic technique. The device requires an isolated dry field; therefore, moisture found in the canal or in the ambient humidity is sufficient to initiate MTA-BIOSEAL hardening;

Mixing: Dual syringe: The dual syringe ensures equal mixing in a 1:1 ratio. Press plunger to extrude material directly onto a glass slab, mixing pad or into the tooth canal. The cement should be used immediately after mixing;

Insertion: Use MTA-BIOSEAL with gutta-percha coating them with a thin layer of the cement and filling the root canal according to the selected technique. MTA-BIOSEAL can be applied in the root canal with a Lentulo spiral or, directly, with an applicator tip adapted to the self-mixing tip;

WARNING: The self-mixing tip must be discarded after use.

Removal of the root canal filling: Use the conventional techniques for the removal of gutta-percha fillings; Working time: 23 minutes.

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