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Indication
Ambulatory tooth whitening based on 6% hydrogen peroxide superior

EN

This kit includes:

- **Smart LED Mouthpiece** (32 LEDS) with charger cable
The Smart LED Mouthpiece has two application modes.
The Blue mode uses a special cold light technology for a standard whitening procedure.
The Blue & Red mode raises the temperature slightly, accelerating the procedure.
- **Whitening pens** (3x 2,5 ml.) with 6% HPS whitening gel
Composition whitening gel: PEG, water, hydrogen peroxide, poloxamer, t-butyl alcohol, etidronic acid, peppermint oil, EDTA, sodium stannate
- **Shade guide**
- **Instruction manual**

Storage:

Avoid direct sunlight and store in cool and dry area. The pens do not need to be stored in the refrigerator.

Terms and conditions:

- Do not use if you are under the age of 18.
- Do not use if you have overly sensitive teeth. The product should only be used to whiten your teeth.
- Do not use the product if your teeth are decaying or loose. The whitening gel will not whiten any existing dental restorations such as fillings, crowns, veneers or bridges.
- Do not use if you are pregnant or lactating.
- Do not use if allergic to any of the ingredients listed above.
- Keep out of reach of children.
- Do not swallow.
- Avoid any contact with soft tissue like the gums and lips by whipping away any excess gel outside the mouthpiece after putting it in the mouth. Excess whitening gel might result in tissue turning red or white. This is nothing to be alarmed about. Just rinse with water and the tissue will return to its normal color after a few days.
- If the whitening gel gets on your face or clothes, you should wash them immediately.
- Always put the cap back on the pen after a whitening treatment to avoid volatilization of the active ingredients.
- Store in a place out of reach for children.
- To be sold only to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.

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EN

Rinse the mouthpiece with water.

A. Brush your teeth and rinse with water.

B. Twist the bottom part of the teeth whitening pen until the gel appears in the bristles.

C. Apply the gel evenly on your teeth, both upper and lower.

DO NOT APPLY TOO MUCH GEL, MORE GEL WON'T SPEED UP THE PROCEDURE OR GIVE BETTER RESULTS!

D. Put mouthpiece in your mouth and softly bite it to turn the light on*.

E. - **Blue Mode** for a 15 minute treatment.

- **Blue & Red Mode** for a 10 minute treatment.

The whitening device will turn off automatically.

F. Take the mouthpiece out.

G. Rinse your mouth and the mouthpiece with water.

Do not eat or drink anything except water for 30 minutes after you finished the treatment.

Frequency:

We advice two whitening sessions per day until the desired result is achieved.

Range of application:

To remove tooth staining from coffee, tobacco, food coloring, aging and other factors that may cause discoloration.

* If you experience any tooth sensitivity after using the Red & Blue Mode, try the Blue Mode.

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