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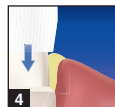
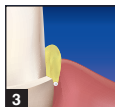
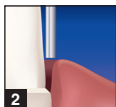
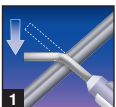
EN Haemostatic paste retraction system

PRODUCT DESCRIPTION:

Traxodent® Haemostatic Paste for Gingival Retraction is a soft paste containing aluminum chloride to provide gingival retraction and hemostasis. The product is topically dispensed through a syringe and a single-use dispensing tip.

INDICATIONS :

Traxodent® has been specifically formulated to provide chemo-mechanical gingival retraction and hemostasis. It is intended for use prior to taking an impression, cementation, cavity preparation or whenever hemostasis and retraction is required.

DIRECTIONS FOR USE:

- 1) Remove the cap from the syringe. Be sure to retain cap for subsequent storage of the material.
- 2) Place annealed leur dispensing cannula onto end of the syringe tip, twist-lock into place. The soft metal of the annealed canula allows bending of dispensing tip for optimal access. A mirror handle can assist in bending tip to desired angle (image 1).
- 3) Rinse and air dry prepared tooth. Extrude Traxodent® paste slowly into the sulcus while maintaining the dispensing tip just above the sulcus and aligned approximately parallel to the axial plane of the tooth preparation. Be careful not to jam dispensing tip into sulcus (image 2).

- 4) Ensure you extrude sufficient material into the sulcus to achieve adequate tissue retraction (image 3).

Note: for subgingival margins and maximum deflection of soft tissue, use a compression cap like Retracap compression cap or similar device, after dispensing Traxodent®. Press Retracap onto prepared tooth. Have the patient bite down or apply pressure to cap if the patient lacks opposing dentition. Optionally, a single strand of retraction cord can be placed prior to dispensing Traxodent® (image 4).

- 5) Allow Traxodent® paste to remain in the sulcus for at least one to two minutes (image 5). Often, the marginal gingiva will grow pale when retraction is achieved.

- 6) After hemostasis has been achieved, the Traxodent® should be removed by thoroughly rinsing using air-water syringe and saliva ejector or suction tip (image 6).

- 7) Examine the treatment site to verify complete removal of the Traxodent® paste prior to taking the impression. Remove canula dispensing tip and dispose of properly after use. Re-cap Traxodent® syringe for storage in sealable foil bag.

CONTRAINDICATIONS:

Do not use on patients with a known allergy to aluminum chloride. Do not use on patients with significant periodontal disease or furcation involvement.

WARNING: Do not use on persons with known allergy to aluminum chloride. Do not freeze.

Adverse events: Allergic type of reactions such as swelling, inflammation, irritation and pain could occur in persons with known allergy to aluminum chloride.

PRECAUTIONS:

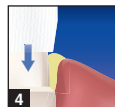
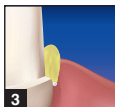
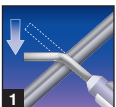
Do not insert the canula dispensing tip into the sulcus. Ensure that all paste is removed before taking impression to avoid inhibition of polymerization of the impression material. The syringe's purpose is Traxodent primary packaging and it is not to be cleaned and reused for any other purpose.

Disposable tips are designed for single patient use only.

Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients and staff.

STORAGE:

Do not freeze. Store at room temperature <28°C / 82°F. Contains syringes and applicator tips. The Traxodent® safety data sheet is available at www.itena-clinical.com.

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TRAXODENT® is a trademark of Premier® Dental Products Company.
Plymouth Meeting, PA 19462 U.S.A.