TOTALCEM

AUTOMIX / DUAL-CURE

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TOTALCEM

AUTOMIX / DUAL-CURE

Self-etching / self-adhesive resin cement for metal. **EN**

TOTALCEM is a dual curing permanent adhesive resin cement specially formulated for metal cementation. TOTALCEM does not require etching, priming or bonding. TOTALCEM is self and light curing. The "Automix" system provides a rapid, consistent and homogeneous mixture which can be dispensed directly into the restoration or tooth cavity.

INSTRUCTIONS FOR USE

- 1) Remove the provisional restoration. Note: Use only temporary cements that are eugenol-free. The use of eugenol-containing temporary materials can inhibit the polymerization process of the TOTALCEM.
- 2) Rinse thoroughly
- 3) Air dry the preparation (do not desiccate).
- 4) Try in the restoration to ensure proper fit.
- 5) Extrude a small quantity of material in order to align both pastes before use. Place a mixing tip on the Automix syringe.
- 6) Dispensing of the material can be done as follows:
 - a. **Inlay/Onlay**
Mount the supplied intra oral tip onto the mixing tip and dispense the cement directly into the cavity preparation covering all surfaces. Place the restoration gently onto the preparation allowing the cement to flow from all sides.
 - b. **Crown**
Dispense the cement directly onto the restoration, covering all surfaces. Place the restoration gently onto the preparation allowing the cement to flow from all margins.
 - c. **Post**
Mount the supplied extra-fine intra oral tip onto the mixing tip. Dispense the cement into the canal. Place the post and move it slightly to avoid possibility of trapped air. Allow the cement to slowly flow from the canal space.
- 7) When the restoration is properly set up, remove all excess cement. Excess cement is best removed when it is in the gel state with a scaler or probe. Gel state can be achieved by curing excess with a light curing unit for 2 seconds or allowing to self-cure for 1,5 – 2 minutes after application. For best results, leave to rest for 1,5 minutes before light curing.
- 8) After removal of excess cement, light cure all surfaces including margins for 20 seconds.
- 9) Use finishing strips for proximal areas. Polish margins with polishing discs, cups or points.
- 10) Check occlusion.

Main composition	
Matrix	UDMA
	Bis-GMA
	TEGDMA
Acidic monomer	4-META
Filler	Barium glass
	Fumed silica

Note : Automix tips are for single patient use only.

CAUTION

Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissues. Wash thoroughly with water after contact.

STORAGE

Do not expose to temperature exceeding 24°C/75°F
Keep away from direct sunlight
Refrigerate for maximum shelf life
Do not freeze
If refrigerated, allow the syringe to reach room temperature before use
Do not use after expiration date. See expiry date on the packaging.

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